

MATTRESS CLEANING GUIDE

The following processes is recommended, but should be adapted to comply with local institution protocols. If you are uncertain, you should seek advice from your local Infection Control Specialist.



Step 1

Prepare an appropriate cleaning and disinfectant solution following the manufacturers guidelines




Step 2

Clean mattress cover with an approved cleaning agent using a single use cloth to remove bodily fluids



Step 3

Rinse with clean water using a clean cloth

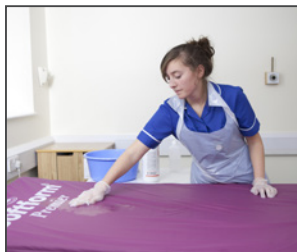

Step 4

Dry thoroughly before re-use following manufacturers guidelines for air drying

Remember:

- Do not use abrasive sponges and cleaners.
- Do use an appropriate chemical agent and consult the mattress user manual for guidelines. Alternatively, contact your local representative.
- Frequent or prolonged exposure to higher concentration disinfectant solutions may prematurely age the fabric cover of mattresses.